Vectra® Basic Air Walker

Vectra® Premium Air Walker

Short and Tall
Klein und Groß
Corto e lungo
Court et haute
Corta y alta


Fitting, Use, and Care Instructions
Anleitung zu Anpassung, Gebrauch und Pflege
Istruzioni per l'adattamento, l'uso e la manutenzione
Mode d'emploi, mise en place et consignes d'entretien
Instrucciones de ajuste, uso y cuidado

Breg, Inc.
2885 Loker Ave. East
Carlsbad, CA 92010 U.S.A.
P: 800-321-0607
F: 800-329-2734
www.breg.com
AW 1.90333 REV D 0812

E/U authorized representative
MDSS GmbH
Schiffgraben 41
D-30175 Hannover
Germany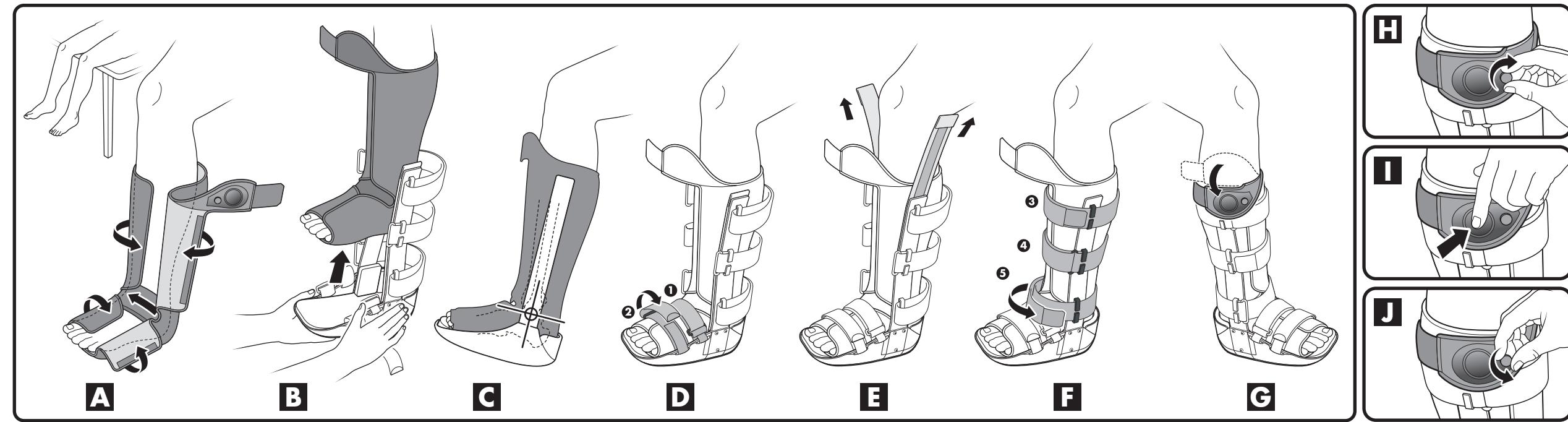

WARNINGS

WARNING: CAREFULLY READ USE/CARE INSTRUCTIONS AND WARNINGS PRIOR TO USE.
WARNING: DO NOT REMOVE WALKER BOOT UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.
WARNING: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: IF YOU EXPERIENCE ANY SKIN IRRITATION OR ADVERSE SKIN REACTION WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR HEALTH CARE PROFESSIONAL.
WARNING: TAKE APPROPRIATE PRECAUTIONS WHEN WALKING ON SLIPPERY SURFACES, UNEVEN TERRAIN, OR ANY OTHER PROPERTY CONDITION WHERE THE EXERCISE OF CAUTION APPEARS NECESSARY.
WARNING: DO NOT OPERATE A MOTOR VEHICLE OR HUMAN POWERED VEHICLE WHILE WEARING THIS WALKER BOOT.
WARNING: DO NOT OVER INFLATE THIS PRODUCT. SIGNS OF OVER-INFLATION INCLUDE, BUT ARE NOT LIMITED TO: DECREASED CIRCULATION (NUMBNESS AND TINGLING), SKIN SENSATION CHANGES, SKIN COLOR CHANGES, AND AN INCREASE IN PAIN.
WARNING: PLEASE CONSULT YOUR HEALTH CARE PROFESSIONAL BEFORE USING THIS PRODUCT ABOARD AN AIRPLANE AS THE INCREASE IN AIR PRESSURE CAN ADVERSELY AFFECT ITS NORMAL FUNCTION AND FURTHER INJURY.
WARNING: DEVICE MAY NEED TO BE RE-INFLATED PERIODICALLY.
CAUTION: FOR SINGLE PATIENT USE ONLY. OUTER SOLE MAY DETERIORATE WITH MULTIPLE USES.

Initial application by a medical professional only!

- A** Apply liner to leg. With patient seated, leg(s) bent, and feet off floor, position tongue pad as needed for maximum comfort, close liner snugly.
- B** Slide walker onto leg over the liner straight up directly under foot.
- C** Position walker with uprights centered over the malleoli (ankle bones).
- D** Fasten straps. Start with straps 1 and 2 closest to the toes (Straps may be trimmed to desired length by removing tab at end of straps, trimming and re-affixing tab).
- E** Remove cardstock quick-pulls from the uprights and discard. Attach uprights to the liner. Press firmly.
- F** Fasten straps. Fasten remaining straps 3 through 5.
- G** Fold down inflation bulb and affix in preferred location (bulb position is variable and dependent on calf circumference).
- H** Close release valve clockwise (to the right).
- I** Inflate air bladders by pressing on inflation bulb to desired pressure for maximum comfort and stabilization.
- J** To release the air, twist the release valve counter-clockwise (to the left).
To re-inflate the air bladders, twist the release valve clock-wise and press the inflation bulb to desired pressure.

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