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
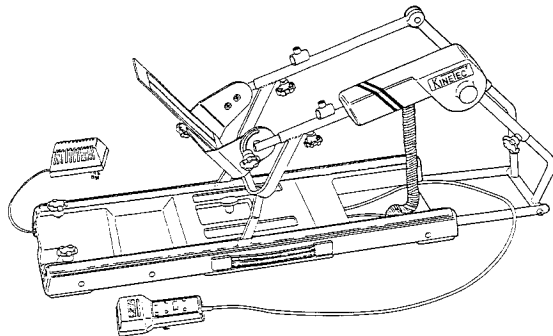
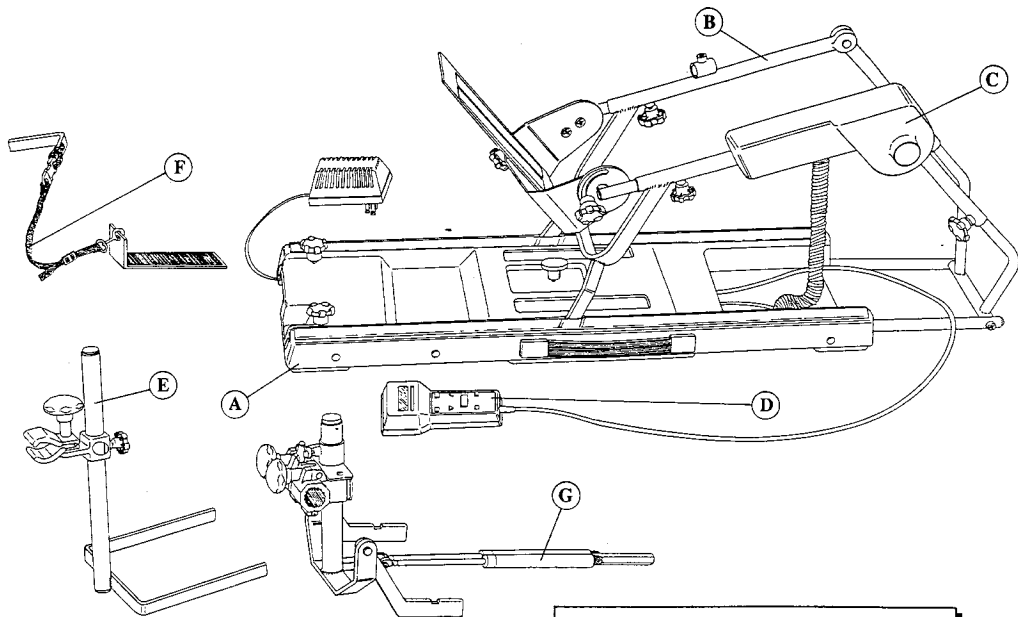
COGEMO reserves the right to effect technical modifications.

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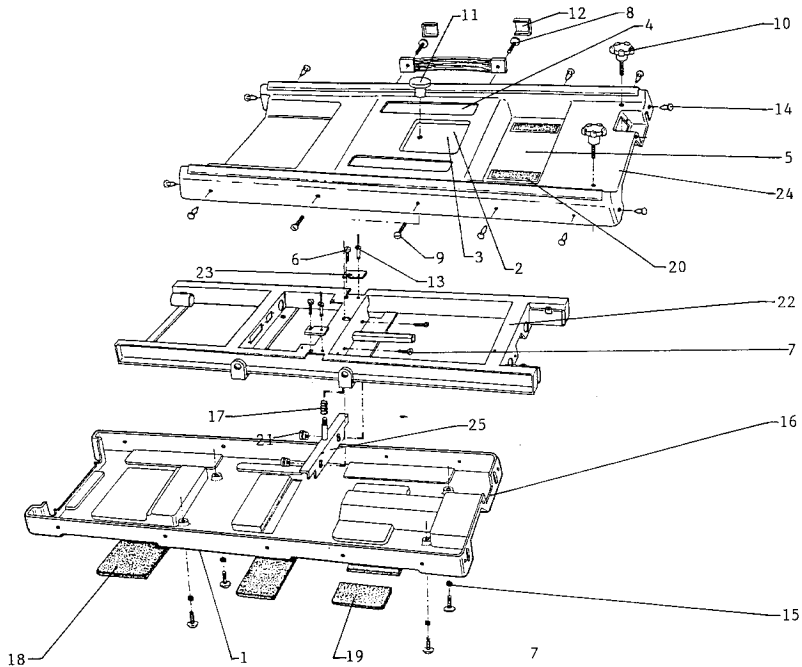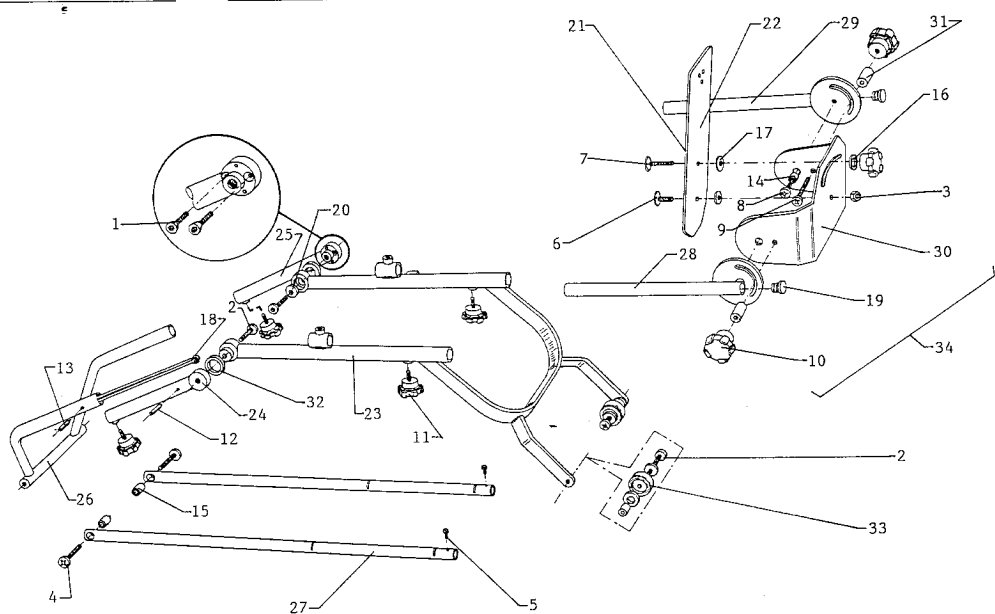
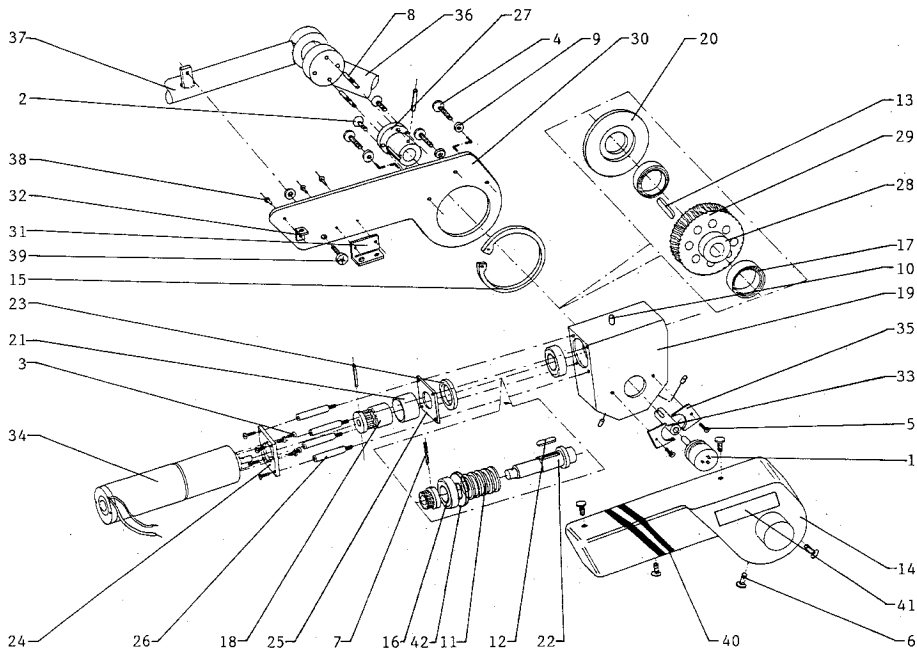
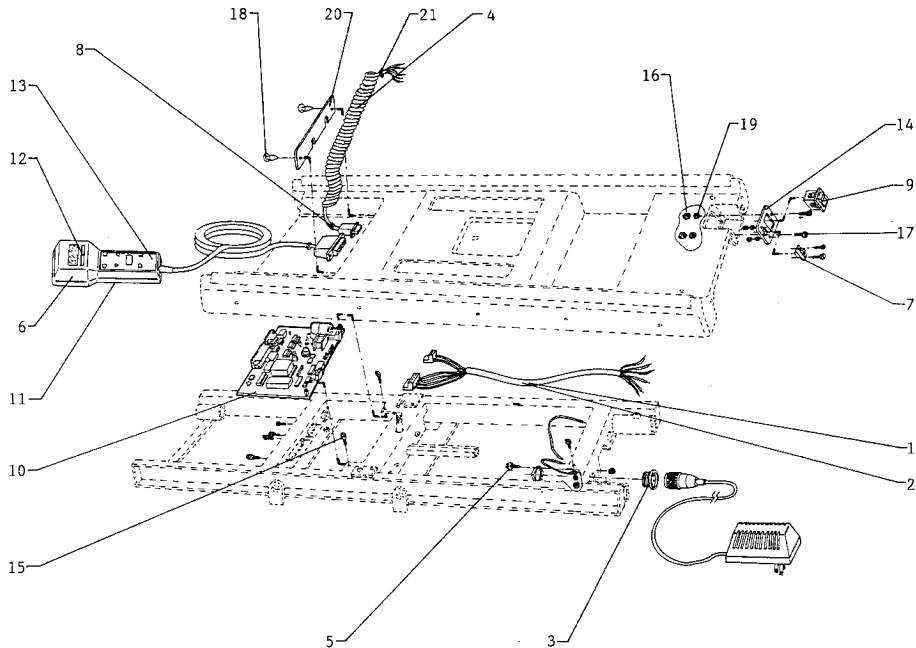
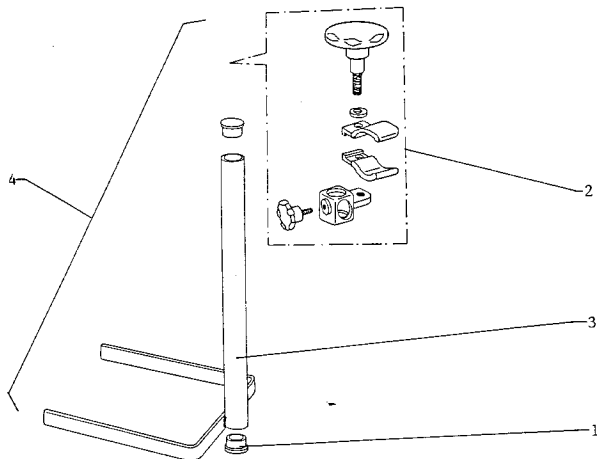
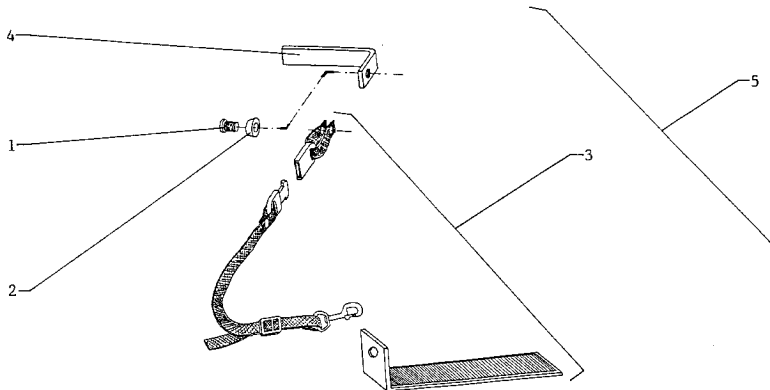
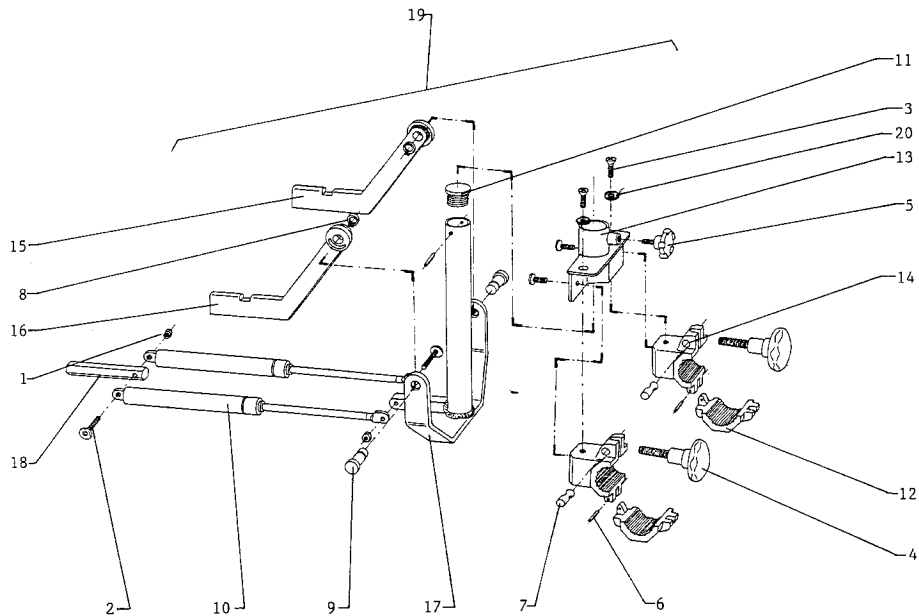
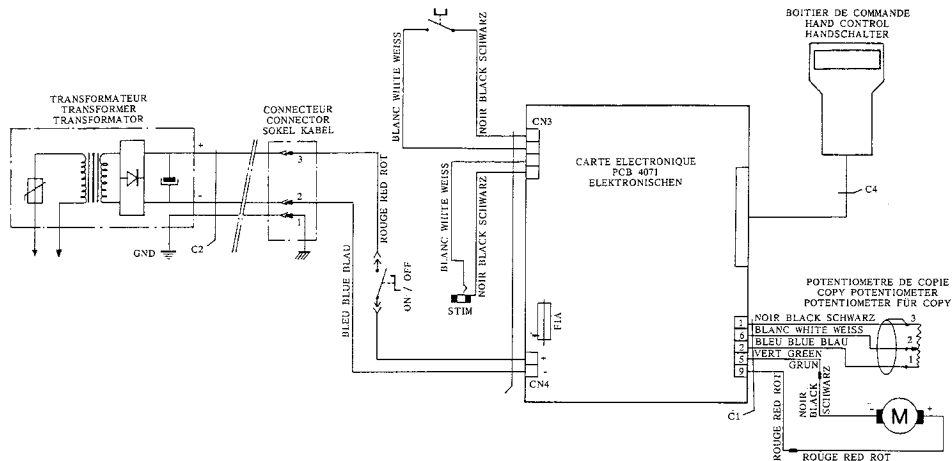
H SCHEMA ELECTRIQUE H WIRING DIAGRAM H SCHALTPLAN

CONTINUOUS PASSIVE MOTION MACHINES

SERVICE POLICY For Service and Repair of Products

1. Products Serviced and Repaired by SMITH & NEPHEW RICHARDS INC.:

Subject to the provisions set forth herein, RICHARDS will service and repair any products manufactured and sold by RICHARDS that are under warranty. Unauthorized repair of products under warranty will void the warranty.

2. Procedure for Service and Repair:

In order for the product to be serviced or repaired, the product must be forwarded at the customer's expense to RICHARDS.

3. Costs of Service and Repair:

The costs of the service and repair rendered by RICHARDS will be based upon the then prevailing hourly labor rate and list price for parts on the date RICHARDS performs the requested service and repair. RICHARDS reserves the right to change the hourly labor rate and list price for parts at any time, without prior notice to the customer.

4. Warranty of Service and Repair:

RICHARDS warrants the service and repair of each product to be free from defects in material and workmanship under normal use and service for the warranty period set forth below and subject to the provisions hereof.

As to the performance of service and repair, the obligation of RICHARDS under this warranty is solely limited to the following:

If during the 30 day period subsequent to any service or repair of any product by RICHARDS, the product again fails to properly function; and

if such malfunction, in the judgment of RICHARDS, is solely the result of improper or inadequate service or repair, RICHARDS will again service or repair the product at no charge to the customer for labor or parts. The cost of any mailing or freight charge for return of the product will be the responsibility of the customer. The 30 day period will begin on the date that the product leaves RICHARDS.

The foregoing warranties are in lieu of all other warranties express or implied, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose or warranties arising from a course of dealing or usage of trade.

5. Refusal of Service and Repair:

RICHARDS reserves the right to refuse to service or repair any RICHARDS product forwarded to RICHARDS by a customer for service or repair. Upon such refusal, RICHARDS sole obligation to the customer, will be the return of the product to the customer, at the customer's sole cost and expense.

6. Changes in the Terms of the Service and Repair Policy:

RICHARDS reserves the right to change, amend or modify any or all of the terms under which it renders product service and repair. Customers who have previously received service and repair work from RICHARDS will be notified of any above-mentioned change, amendment, or modification of terms.

LIMITED EXPRESS WARRANTY

SMITH & NEPHEW RICHARDS INC. warrants each new product to be free from defects in material and workmanship under normal use and service, for the warranty period set forth below and subject to provision hereof.

The obligation of RICHARDS under this warranty is solely limited to repairing or replacing on an exchange basis at the election of RICHARDS, any product or part thereof, which is within **two years**, after delivery of the product to the original purchaser, returned to RICHARDS transportation prepaid, and which is determined by RICHARDS to be covered by the warranty. Warranty for reconditioned products or parts thereof is limited to **90 days**.

The warranty will not apply to any product which has been repaired or altered outside of the factory of RICHARDS, which RICHARDS determines in its sole discretion, has been subject to misuse, negligence, or accident.

The liability of RICHARDS, pursuant to the above-mentioned warranty, will exist only if the products are operated, cleaned and properly lubricated and otherwise used and maintained in accordance with the instructions furnished by RICHARDS and have been subjected to normal use for the purpose for which the goods were designed.

The foregoing warranties are in lieu of all other warranties express or implied, including, but not limited to, the implied warranties of merchantability and fitness for a particular purpose or warranties arising from a course of dealing or usage of trade.

RICHARDS will not be liable for special, indirect, consequential damages or other damages. The remedies set forth herein are exclusive, and the

liability of RICHARDS with respect to any contract sale, rental, of anything done in connection therewith, whether in contract, in tort, under any warranty, or otherwise, will not, except as expressly provided herein, exceed the price of the product on which such liability is based.

Any affirmation of fact, description of goods, or sample or model referred to in any product brochure, specification sheets or the like, whether or not the same relate to production or capability of the products to perform, do not form part of the basis for the bargain but are merely made in the course of the negotiations of the parties and are not to be construed as warranties unless specifically identified as such.

RICHARDS makes no warranty of any kind, express or implied, except as set forth above, and the purchaser shall be responsible for all risk and liability to any third party resulting from or arising from the use of said products, equipment or goods. RICHARDS neither assumes, nor authorizes any person to assume for RICHARDS, any other liability or obligation in connection with the sale or use of the products sold.

FOR THE WARRANTY TO BE EFFECTIVE, THE PURCHASER MUST COMPLETE AND RETURN THE WARRANTY CARD PROVIDED BELOW WITHIN 15 CALENDAR DAYS FROM THE DATE OF PURCHASE.