

# 9081 Hip CPM

## Assembly/Setup Instructions

# H-Base and Support Assembly

- Remove the nut and washer from the upright support and attach the upright support to the red H-Base using the flat wrench and tighten firmly. The black knob on the upright is attached with the hex wrench provided.



# H-Base and Support Assembly

- The Upright should point towards the long end of the H-base





# Main CPM Assembly

- Remove the flat nut from the bottom of the CPM unit and place the unit on the upright support, replace the nut and tighten firmly



# Main CPM Assembly

- Place the thigh length adjuster on the unit and tighten the knob just enough to keep it from moving

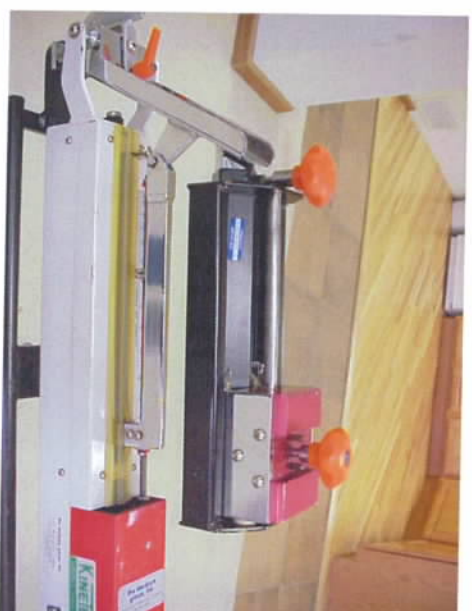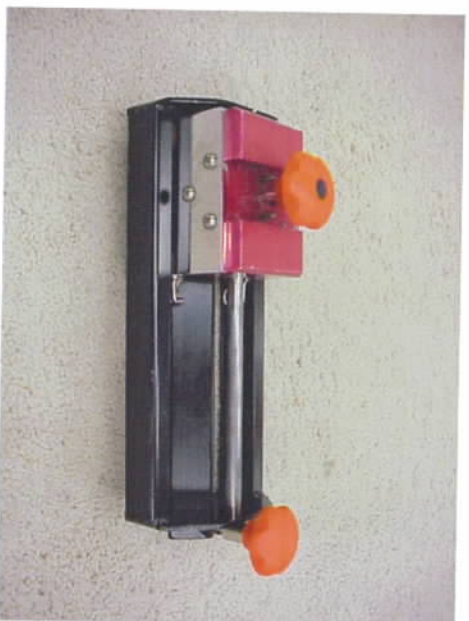



# Leg Cradle Assembly



# Leg Cradle Assembly

- The L-bracket will attach to either side of the leg cradle and will be determined by which leg is to be manipulated



# 9081 Hip CPM

## Assembly/Setup Instructions



# H-Base and Support Assembly

- Remove the nut and washer from the upright support and attach the upright support to the red H-Base using the flat wrench and tighten firmly. The black knob on the upright is attached with the hex wrench provided.



# H-Base and Support Assembly

- The Upright should point towards the long end of the H-base





# Main CPM Assembly

- Remove the flat nut from the bottom of the CPM unit and place the unit on the upright support, replace the nut and tighten firmly



# Main CPM Assembly

- Place the thigh length adjuster on the unit and tighten the knob just enough to keep it from moving

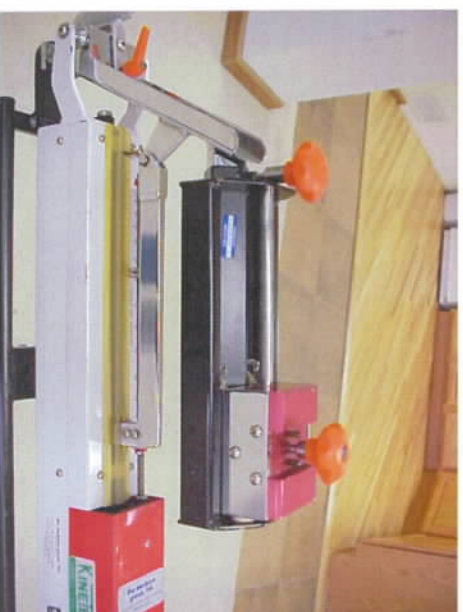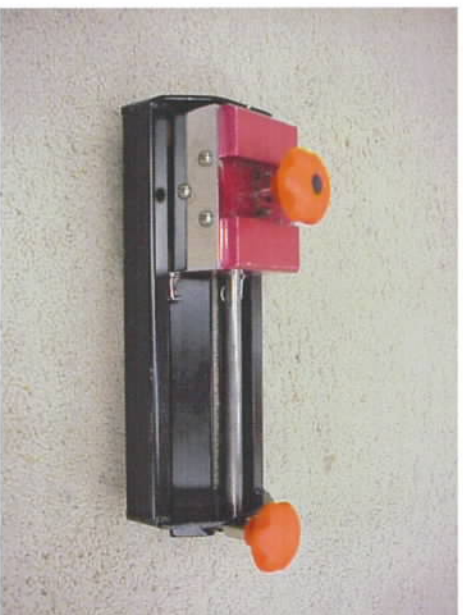



# Leg Cradle Assembly





# Leg Cradle Assembly

- The L-bracket will attach to either side of the leg cradle and will be determined by which leg is to be manipulated





# Leg Cradle Assembly

- Attach the L-Bracket to the leg cradle



# Leg Cradle Assembly

- Attach the leg cradle to the main CPM by loosening the knob on the thigh adjuster until the L-bracket will slip over the locating pins and tighten the knob snugly

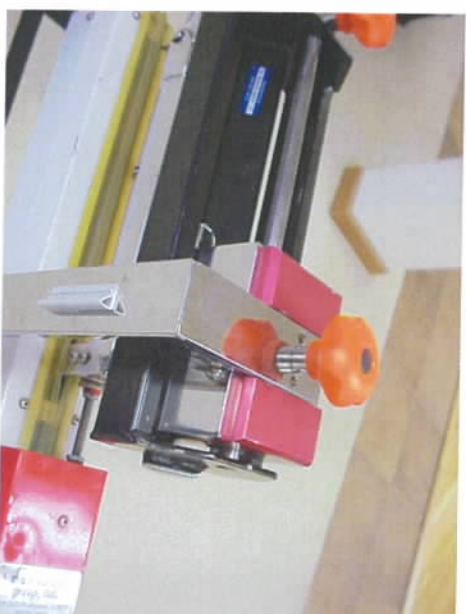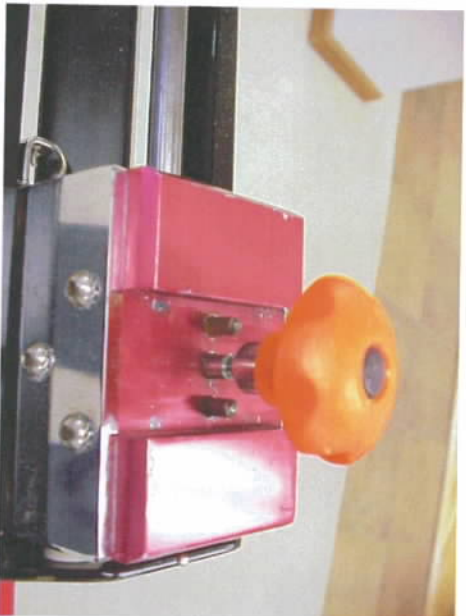

# Leg Cradle Assembly





# Cable Connections

- There are 3 cables that connect to the control box
- The power cable
- The motor cable
- The patient control cable





# Cable Connections

- The patient control cable has the pushbutton at one end, connect it to the back of the control box. This will be on the left side of the back of the control box.

# Cable Connections

- The motor control cable has 7 male pins on one end and 7 female pins on the other, connect the corresponding ends to the control box and the connection on the end of the main CPM

# Cable Connections

- The power cable plugs into the control box and into a standard 115/120 volt outlet

# Pad Kit

- Attach the pad kit to the leg cradle

# Final Adjustments

- Turn unit on and run unit to 15 degrees
- Place patient into padded leg cradle until the hip joint is centered with the carriage pivot point



# Final Adjustments

- Loosen the thigh adjustment knob and adjust leg cradle until the knee joint of the cradle aligns with the patients knee and tighten knob firmly

# Final Adjustments

- Loosen the foot plate adjustment knobs and slide foot plate in or out for proper fit

# Final Adjustments

- To set the machine for flexion/extension, position the adjustors on the threaded rod and lock in place with companion adjustors