
























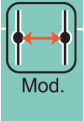

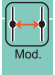
























SPECTRA S1 QUICK REFERENCE GUIDE

TASK	ACTION	NOTE
LOCK / UNLOCK HAND CONTROL	Simultaneously press the  and  keys	Hand control remains locked when the unit is switched ON/OFF at black switch
MANUAL SET UP MODE	<p>Switch the CPM ON at black switch</p> <p>Press  on hand control to run the machine.</p> <p>To change the <u>EXTENSION</u> limit, press  <small>limit</small></p> <p>Then use  or  key to increase or decrease extension ROM</p> <p>To change the <u>FLEXION</u> limit, press  <small>limit</small></p> <p>Then use  or  key to increase or decrease extension ROM</p> <p>To change <u>SPEED</u>, press SPEED</p> <p>Press  or  to increase or decrease speed</p> <p>To change <u>FORCE</u>, press FORCE</p> <p>Press  or  to increase or decrease force</p> <p>To change <u>TIMER</u>, press TIMER</p> <p>Press  or  to increase or decrease time</p>	<p>CPM will run between the preset ROM settings.</p> <p>The extension value will blink. (Make  or  changes within 3 seconds)</p> <p>The flexion value will blink. (Make  or  changes within 3 seconds)</p> <p>Value will blink. To validate new changes, wait 3 seconds.</p>
SESSION TIME 	Press Session Time key  to show the running time of the session	Session time is reset each time CPM is switched on at black switch
WARM UP MODE 	<p>Unit must be stopped </p> <p>Press warm up mode key </p> <p>Press start </p>	SPECTRA starts at 70% of full ROM and increases 5% of the range every other cycle until preset ROM is reached.

TASK	ACTION	NOTE
<p>MODULATION MODE</p> 	<p>Unit must be stopped </p> <p>Press Modulation mode key </p> <p>Press and hold down  to go into FLEXION until desire angle has been reached, then save degree by pressing  immediately.</p> <p>OR</p> <p>Press and hold down  to go into EXTENSION until desire angle has been reached, then save degree by pressing  immediately.</p> <p>Press start  to operate CPM with new ROM limits.</p>	<p>Used to ensure CPM ROM is within pain limits.</p>
<p>BY-PASS MODE</p> 	<p>Unit must be running (press start) </p> <p>Press By-Pass mode key </p> <p>Press and hold down  if the machine is going into FLEXION - CPM will bypass preselected flexion range. Save new ROM by pressing  immediately.</p> <p>OR</p> <p>Press and hold down  if the machine is going into EXTENSION - CPM will bypass preselected extension range. Save new ROM by pressing  immediately.</p>	<p>Used to bypass preset ROM limits.</p>
<p>PROGRAM MODE</p>	<p>Turn machine OFF at black switch</p> <p>Hold down  and  buttons at the same time whilst turning unit on at black switch. Program will appear on the hand control. Release the  and  buttons once program message appears.</p> <p>Select one of the 16 programs by pressing  or </p> <p>SELECT MODE:</p> <p><i>WARM UP</i> - press  once <i>MANUAL SET UP</i> - press  twice</p> <p>Set ROM, force, pause time and speed as per normal. Then press Program Key once all settings have been made.</p> <p>To save program, press  To clear program, press </p> <p>Turn off machine and turn it on again to access stored programs.</p>	