

1 Connect the output connector and the power supply to the back of the unit.



- Connect the power cord to the power supply input connector and connect the opposite end of the power cord to a grounded wall outlet. The unit will power-up and the Start screen will be presented.

2 Coolant Reservoir Fill Location

- Unscrew and remove the reservoir cap and fill the reservoir to the bottom of the reservoir neck. Close the cap tightly.



Reservoir cap/fill location

- Use Caution not to overfill the Reservoir.

3 Touch Screen Illuminated Display



4 Start-up Steps

- Place unit upright on level surface 1-foot from any obstruction that could restrict airflow through the unit.
- Connect the output connector and the power supply to the back of the unit as described in 1 above. **Follow steps 3-12 on the touch screen display for a complete setup guide prior to beginning any treatment.**
- Connect the power cord to the power supply input connector and connect the opposite end of the power cord to a grounded wall outlet. The unit will power-up and the START screen will be presented.
- Apply the prescribed therapy wraps to the indicated portions of your body as described on the wrap instructions contained in the wrap packaging.
- Unscrew and remove the reservoir cap from the top of the unit. Using the coolant bottle supplied with the therapy system, fill the reservoir to the bottom of the reservoir neck as described in 2 above. Close the cap tightly.
- Connect the clear and gray hoses from the umbilical hose to the therapy wrap used for treatment. On the front of the unit, connect the clear and gray hoses from the umbilical hose. If DVT compression wraps are to be used, connect the wrap connectors to the DVT ports of the unit.
- If the device has been pre-programmed, then by selecting "START THERAPY" on the Start screen as illustrated in 3 above, the pre-programmed therapy is initiated and the Main Therapy screen will appear.
- To stop therapy, select "STOP THERAPY" on the Main Therapy screen.

Please see the User Manual for additional information, indications and contraindications for use.