


1 Plug the supplied power cord into the unit and into a grounded AC outlet.

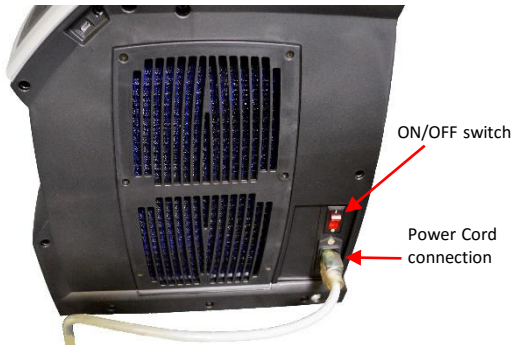

- Turn ON the unit. The unit will power up and present the START menu.


2 Coolant Reservoir Fill Location

- Unscrew and remove the reservoir cap and fill the reservoir to the bottom of the reservoir neck. Close the cap tightly.



- Use Caution not to overfill the Reservoir.

3 Therapy Wrap Connections and Touch Screen Illuminated Display

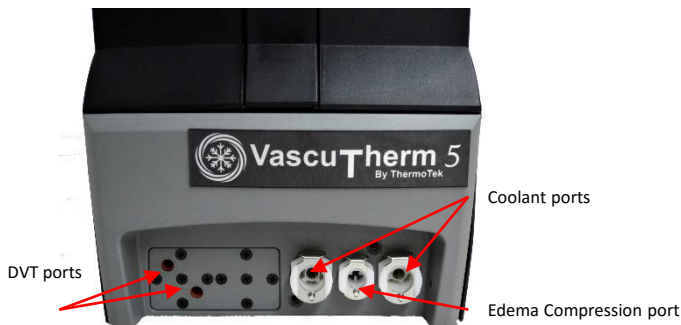

4 Start-up Steps

1. Place unit upright on level surface 1-foot from any obstruction that could restrict airflow through the unit.
2. Connect the supplied power cord to the unit as described in 1 above.
3. Turn on the main power switch on the side of the unit.
4. Apply the prescribed therapy wraps to the indicated portions of your body as described on the wrap instructions contained in the wrap packaging.
5. Unscrew and remove the reservoir cap from the top of the unit. Using the coolant bottle supplied with the therapy system, fill the reservoir to the bottom of the reservoir neck as described in 2 above. Close the cap tightly.
6. Connect the umbilical hose to the therapy wrap used for treatment. On the front of the unit, 3 connect the clear and gray hoses from the umbilical hose to the fluid/compression ports. If DVT compression wraps are to be used, connect the wrap connectors to the DVT ports of the unit.
7. If the device has been pre-programmed, then by selecting "START" on the Start screen as illustrated above, the pre-programmed therapy is initiated and the Main Therapy screen will appear.
8. To stop therapy, select "STOP THERAPY" on the Main Therapy screen.

Please see the User Manual for additional information, indications and contraindications for use.